TERUMO SVENSKA Implantatkort

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Terumo Medical Corporation
265 Davidson Avenue, Suite 320
Somerset, NJ
08873 USA

Terumo Europe N.V.
Interleuvenlaan 40
3001 Leuven
Belgium

AU Sponsor:
Terumo Australia Pty Ltd
Macquarie ParkNSW 2113
Australia

Made in USA

- 👤Name of the patient or patient ID. To be filled by the healthcare institution/provider.
- 📅Date of implantation. To be filled by the healthcare institution/provider.
- 📍Name and address of the healthcare institution. To be filled by the healthcare institution/provider.
- Select location where FemoSeal device was implanted.
- Place sticker from device packaging in “Place sticker” location.

To the FemoSeal™ Vascular Closure Device Patient
Please report to the doctor any bleeding, swelling, or bruising around the puncture site, or any discomfort experienced after the puncture is sealed. If you will need another puncture of the vessel in your groin, inform your doctor about your FemoSeal device and show this patient information card. Please keep this card with you for the next 18 months.

To the Physician

This patient has received FemoSeal Vascular Closure System after percutaneous catheterization. The device consists of three resorbable components: An inner seal deployed intra-arterially, an outer locking disc deployed on the outer wall of the artery, and a suture trimmed below skin level.

Be aware that the full absorption of inner seal and outer locking disc occurs in 18 months, and if early puncture is needed, it should be made at least one centimeter proximal to the previous FemoSeal Vascular Closure System. The implanted components of the device are MRI Safe.

Terumo Medical Corporation
265 Davidson Avenue, Suite 320
Somerset, NJ
08873 USA

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